


- Access[®] chemiluminescent technology delivers highly specific and sensitive performance
- Two-step, qualitative immunocapture assay
- Excellent clinical performance: 98.3% relative sensitivity and 99.9% relative specificity to another commercially available method
- 100% agreement on 101 seroconversion samples to another commercially available method
- Time to first result: 35 minutes
- Calibration stability: 28 days

2 step IgM
immunocapture
technique

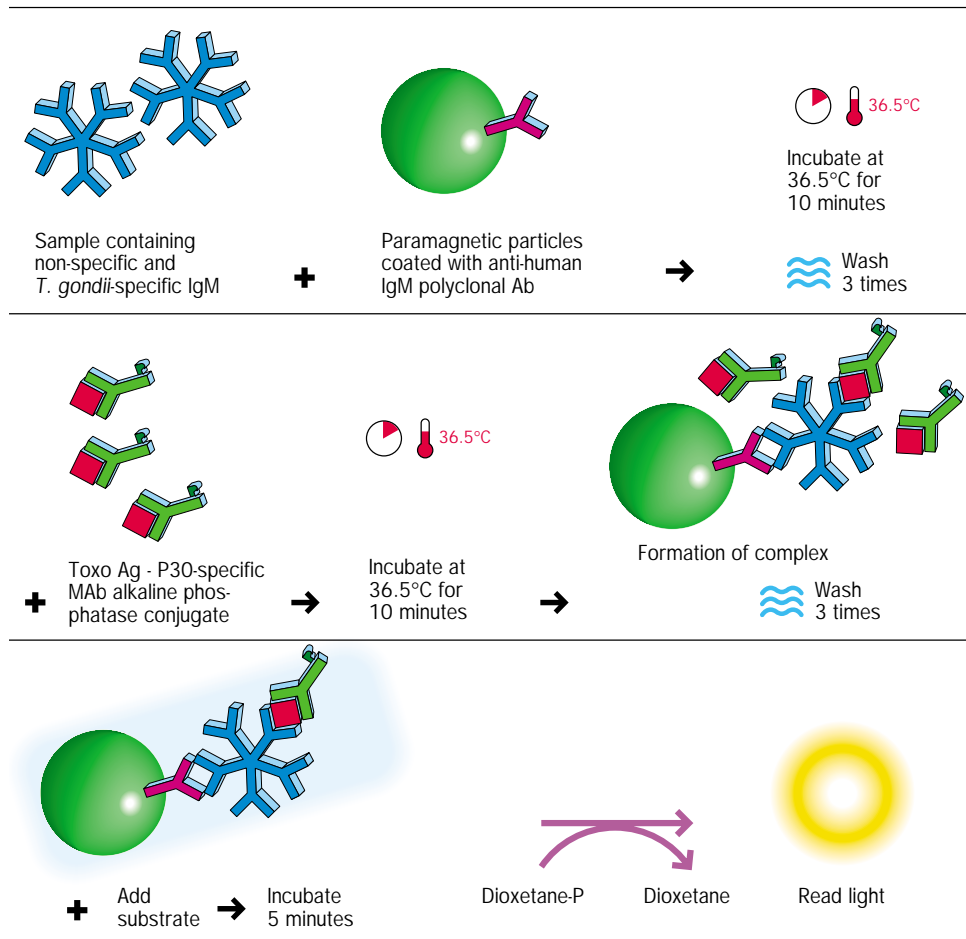


Signal determines the presence or absence of Toxo IgM antibody



Toxoplasma IgM II

Performance Access Toxo IgM II Seroconversion

This seroconversion example shows Access Toxo IgM II having an early and strong S/CO response as compared to EIA (enzyme immunoassay) methods.

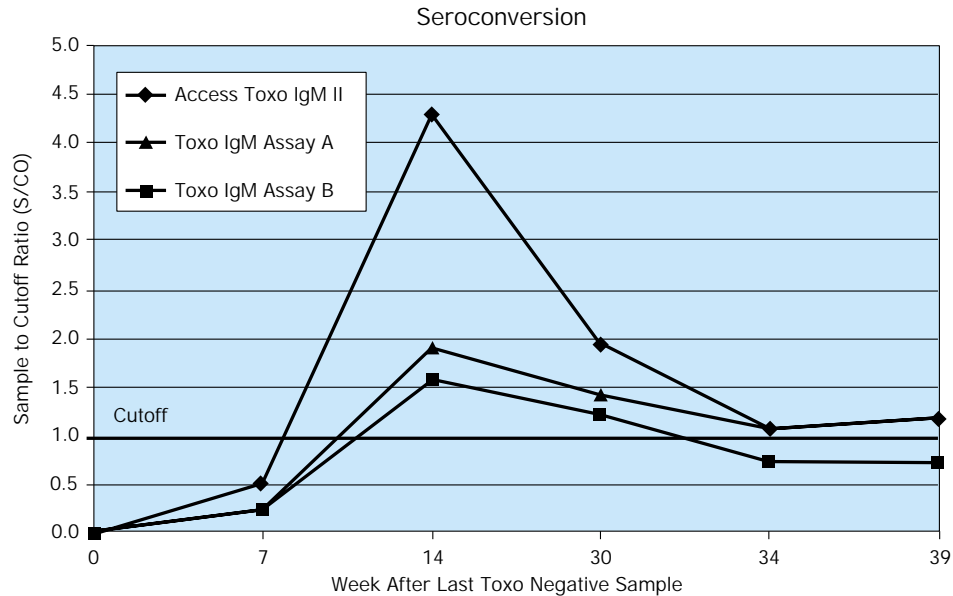

Characteristics

Sample Type/Size	Serum or plasma (EDTA, Citrate, heparin)/10 µL
Time to First Result	35 minutes
Interpretation	Sample to cutoff ratio (S/CO): < 1.0 = nonreactive, 0.8 ≤ S/CO < 1.0 = grey zone, ≥ 1.0 = reactive
Open Pack Stability	28 days
Calibration Stability	28 days

Ordering Information

Access® Toxo IgM II - 2 packs of 50 tests/pack	34470
Access® Toxo IgM II Controls - 2 levels, 1 vial/level of 1.5 mL/vial	34475
Access® Toxo IgM II QC - 2 levels, 3 vials/level of 3.5 mL/vial	34479



Eastern Europe, Middle East, North Africa: Switzerland, Nyon (41) 22 994 0707. Australia, Gladesville (61) 2 9844 6000. Canada, Mississauga (1) 905 819 1234. China, Beijing (86) 10 6515 6028. Hong Kong (852) 2814 7431, 2814 0481. France, Villepinte (33) 1 49 90 90 00. Germany, Krefeld (49) 2151 33 35. Italy, Milan (39) 02 953921. Japan, Tokyo (81) 3 5404 8424. Latin America (1) (305) 380 4709. Mexico, Mexico City (525) 575 6805. Netherlands, Mijdrecht (31) 297 230630. Singapore (65) 6339 3633. South Africa/Sub-Saharan Africa, Johannesburg (27) 11 805 2014. Sweden, Bromma (46) 8 564 85 900. Switzerland, Nyon 0800 850 810. Taiwan, Taipei (886) 2 2378 3456. Turkey, Istanbul (90) 216 309 1900. UK, High Wycombe (44) 01494 441181. USA, Brea, CA (1) 800 352 3433, (1) 714 993 5321.